

## MICRODENT PROVISIONAL IMPLANTS

### Precautions and instructions for use.

These usage instructions should be read before using the implants and materials supplied by IMPLANT MICRODENT SYSTEM S.L.U. as they contain essential information to avoid misuse.

The descriptions below have been written for implant professionals. Therefore, we recommend they receive instructions from an expert professional in the fields, who has received adequate information. Implant Microdent System S.L.U. offers regular training courses on the use of their products.

The user must ensure that the selected product is suitable for the planned purposes and procedures.

These instructions for use are available in pdf format at the following web address: [www.microdentsystem.com/instrucciones-uso](http://www.microdentsystem.com/instrucciones-uso)

To open pdf files you need the free Adobe Acrobat Reader.

#### INTENDED USE

Microdent PIM Dental implants are intended for surgical placement in the upper or lower jaw to provide a means for prosthetic attachment in fully edentulous utilizing delayed or immediate loading and to retain overdentures.

Microdent Dental implants are intended for immediate function when good primary stability is achieved, with appropriate occlusal loading, in order to restore chewing function.

#### DESCRIPTION

Microdent provisional dental implants are designed specifically for support a provisional prosthesis placing itself in further overdenture which will be retained by the own provisional Implant ball leaving definitive Implant without any load till entire Implant healing has been done. This narrow Implant metes tentativeness functions, so it can never be used as a definitive Implant.

The external thread of the implant, of the self-cutting type, together with the geometry of the grooves it has on the apical area of the implant, enables it to be inserted easily, as well as offering high primary retention.

Microdent provisional dental implants are made of grade 5 pure titanium, in accordance with ISO 5832-3:1999 standard, with a surface treatment to achieve greater roughness in the entire osseointegration area.

The nominal length of implants agrees with the length of threaded area.

The plastic cap that accompanies to implants is made in PP thermoplastic material.

The implants are available in different lengths: 8, 10, 12, 14 and 16 mm.

#### MICRODENT PROVISIONAL IMPLANT PRESENTATION.

##### Packaging and sterility

Microdent provisional dental implants undergo an exhaustive manufacturing, control and cleaning process before packaging, in a sterile area, being subsequently sterilized by gamma irradiation.

The implant has on disposal a little hexagon in upper part where keys are applied for its placing.

The plastic support is inside a container that maintains sterility through a metal heat-sealed operculum. The immunity and protection of this sterile barrier system is by means of a threaded cap that seals the outside packaging and guarantees that the contents have not been handled.

Both the external packaging and the internal container have a label giving the batch number, size and model of implant and the use by date.

For correct follow-up of required product traceability, this label should be adhered to or its information recorded in the patient's medical record.

Implant Microdent System S.L.U. will accept no responsibility for the re-sterilization of implants, irrespective of the method used or the person who undertook the procedure. An implant that has already been used or is not sterile must not be fitted, under any circumstances.

#### PLANNING AND LOAD

Before proceeding with the surgical part of implant rehabilitation, it is the responsibility of the user to carefully plan the process.

To maximize load capacity, it is recommended, whenever possible, to use implants of the greatest number possible.

To help with selecting the implants required for rehabilitation, Implant Microdent System S.L.U. has issued x-ray templates reference PPQPIM.

Angles of more than 30° over the vertical of the implant should be avoided.

#### CONTRAINDICATIONS

Serious internal medicine problems, bone metabolism disorders, uncontrolled coagulation disorders, uncooperative or unmotivated patients, alcoholism or drug addiction, psychosis, functional disorders resistant to treatment present for some time, xerostomy, weakened immunological system, illnesses requiring regular use of steroids, allergy to titanium, uncontrolled endocrine disorders.

- Relative contraindications:

- Previously irradiated bone, diabetes, anticoagulant medication / hemorrhagic diabetes, bruxism, parafunctional habits, unfavorable bone anatomy, addiction to tobacco, uncontrolled periodontitis, maxillary illnesses and alterations of the oral mucous membrane susceptible to treatment, pregnancy, poor oral hygiene.

- Local contraindications:

- Insufficient bone available or inadequate bone quality, local remains of roots.

#### SIDE EFFECTS, INTERACTIONS AND COMPLICATIONS CAUSED BY DENTAL IMPLANTS.

Activities requiring great physical effort should be avoided during the period immediately after the insertion of dental implants.

The following are some of the possible complications after fitting dental implants:

- Transitory problems:

Pain, inflammation, difficulty in speaking, gingivitis.

- Longer lasting problems:

Chronic pain associated to the dental implant, permanent paresthesia, dysesthesia, bone loss in the maxillary crest, localized or systemic infections, oroantral or oronasal fistulas, harmful influence on neighboring teeth, irreversible damage to neighboring teeth, aesthetical problems, damage to the nerve, exfoliation, hyperplasia.

It is the responsibility of the user, communication with patients of possible complications that may arise. Before any symptoms or discomfort the patient must go to consulting your practitioner.

#### WARNING

In case that implants do not exhibit good stability is not recommend immediate load.

If the implant comes into contact with contaminants, especially blood and/or saliva, it must not be reused, as there is no guarantee they can be eliminated, even if cleaned and sterilized, as these contaminants can transmit diseases such as AIDS, hepatitis or STDs.

Another reason for not reusing the implant is the possible damage it may have suffered to its geometry due to inadequate handling.

Electro surgery is not recommended due to the conductivity of the dental implants.

Before proceeding with the surgery, it is the user's responsibility to check the implant container to ensure it is the right product for the patient. Implant Microdent System S.L.U. recommends having replacement products available.

During the intraoral use of our products the user must take the necessary precautions to avoid the patient aspirating any part of them.

#### SURGICAL PROTOCOL

##### Preparing the alveolus

These recommendations are to be taken as a general guide and should be personalised by the professional according to the type of bone where the implant is to be fitted.

The depth of the alveolus should be in accordance with the length of the implant, the use of depth marks (6 / 8 / 10 / 12 / 14 / 16 / 18 mm) displayed by the burrs distributed by Implant Microdent System S.L.U. is recommended. Osteotomy procedure: use drill F150 in optimum cutting/cleaning condition, carry out intermittent perforation and ensure good irrigation of the burr to avoid overheating the bone.

We recommend regular rotation of surgical drill to avoid using drill in bad condition or cutting incorrectly.

Since the external irrigation only deepens between 2 and 3 mm, once the drill is inside the surgical socket, it is recommended that every 3 or 4 seconds the drill be removed from the alveolus, always moving and without stopping irrigation, for avoid excessive heating of the bone

In excessively hard bones, perform the normal milling protocol, inserting and completely removing the drill from the socket, to facilitate the replacement of the serum and lower the temperature of the same.

**Fitting the implant**

Open the implant container by unscrewing the plastic cap and breaking the safety seal of the packaging. Remove the metal operculum - the sterile barrier of the packaging system.

Deposit the contents of the container on a clean surface; you will see a plastic support on which the implant is fitted with its metal implant holder, and the transport cap for the manual insertion of the implant. Also, it will leave the plastic cap for the prosthesis retention.

**Manual insertion of the implant**

If the insertion is to be manual, remove the transport cap from the plastic support and place it on the metal implant holder with the implant.

To extract the implant from the plastic support, press the support lightly in the direction of the opening where it is housed. The implant is joined to the transport cap; this helps to place it in the patient's mouth without having to touch it. Insert the implant into the osteotomy and begin securing the implant by turning it clockwise.

If on inserting the implant in the osteotomy (on the first threads) you notice having to use more force than usual, we recommend removing the implant from the alveolus and increasing its size with a larger burr. Forcing the implant could cause damage.

Once the implant is 2/3 inserted in length, finish the insertion by using the extension adaptable to the metal implant holder reference ACC44 or ACL44 and if this is not possible, due to particular space conditions, remove the metal implant holder and finish the process with the adaptor for the corresponding securing of the model of implant being inserted.

We recommend using an insertion torque not greater than 40 Ncm and in no case exceed an insertion torque of 50 Ncm. to avoid the appearance of aseptic necrosis and implant loss. To control torque, the torque-measuring ratchet, reference LDR1070, is recommended

**Inserting the implant mechanically**

If the implant is to be inserted using a micromotor, insert the adaptor reference LC44 that was previously fitted to the hand piece, over the metal implant holder.

To extract the implant from the plastic support, press the support lightly in the direction of the opening where it is housed.

Move the implant to the osteotomy and proceed inserting it, regulating the speed of the micromotor to obtain maximum control over the process.

We recommend using an insertion torque not greater than 40 Ncm and in no case exceed an insertion torque of 50 Ncm. to avoid the appearance of aseptic necrosis and implant loss.

In both forms of implant insertion, soft tissues must be prevented from invading the alveolus, since they harbor a wide variety of bacteria.











It is advisable to observe during the insertion that said alveolus is completely filled with blood clot, in this way the implant will be impregnated by surface tension and blood cells.

**PROSTHETIC ABUTMENTS**

Microdent Provisional implants does not require of prothetic abutments with the exception of the plastic cap that is provided next to implants.

**LABELLING SYMBOLS**

The product label will give the following information:


	Single use only.		Do not use if the package is damaged.
	Reference number.		Use before expiration date.
	Please follow the instructions for use.		Sterilized by irradiation.
	Batch number.		Manufacturer.
	Precaution consult the warnings.		Global trade item number.

**LIABILITIES, SAFETY AND GUARANTEE**

If the sterility packaging of the implant is damaged during delivery, Implant Microdent System S.L.U. will replace it free of charge. This will not be the case if there is evidence of product manipulation.

The guarantee applies to Implant Microdent System S.L.U. implants as long as they have been used in accordance with the indicated instructions of use.

To proceed with returns of products mentioned in these instructions for use, please follow the regulations of our sales and supply conditions.

 Implant Microdent System S.L.U.  
 Pol. Ind. Can Magre C/. Carles Buïgas, 1  
 08187 Sta. Eulalia de Ronçana (Barcelona)

