





NOTE: To insert a new retainer or Micro-Loc abutment, start with step 1.
To remove a nylon retainer, start with step 2.


1. To fit a new nylon retainer into the overdenture cap, disengage the cap removal tool (SLTM03) from the cap seating tool (SLTM01) and use the end of the cap seating tool (SLTM01), as shown in the drawing.

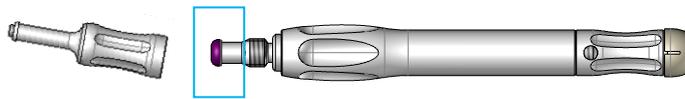


The plastic sleeve (SLTM05), located at the tip of the abutment driver (SLTM02), is designed to secure the Micro-Loc abutment to the screwdriver and facilitates its vertical position without releasing it, helping to bring the abutment to the patient's mouth.

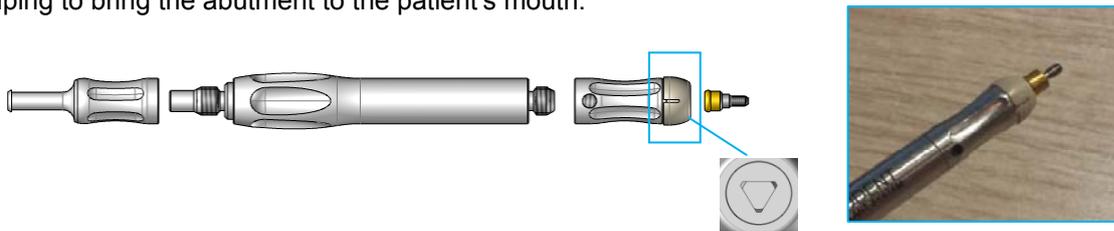


2. To remove a nylon retainer attached to a metal cap, loosen the cap removal tool (SLTM03) by making 2 full left turns (a gap is then visible).

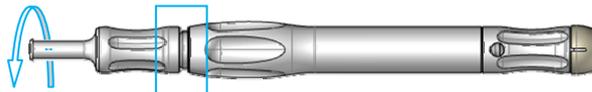

Insert the tip into the retainer/metal cap set and push straight in to the bottom of the nylon retainer. Then tilt the key until the sharp edge of the tip captures the retainer, pull backwards and remove it from the cap.

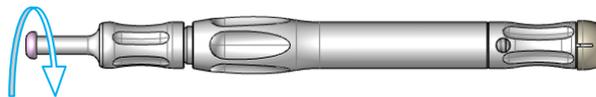

To discard the removed retainer, aim the key down and away from you and turn the cap removal tool (SLTM03) clockwise back into the cap seating tool (SLTM01). This will activate the removal of the cap seating annex (SLTM04) and will expel the damaged nylon retainer from the tip end.

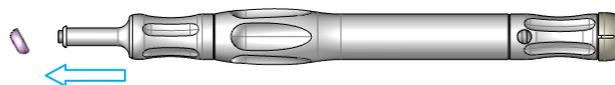

NOTE: Do not attempt to use the cap removal tool (SLTM03) to place retainers as they may be damaged.