


IMPLANTS PACKAGING



1. Open the implant threaded cap counter-clockwise.




2. Remove the aluminum cap from the container by the small flap.

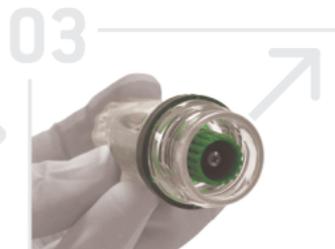

3. Extract the implant's container. The plastic implant-holder contains the cover screw.



4. Use the plastic implant-holder and set it in the square of the metallic implant-holder.

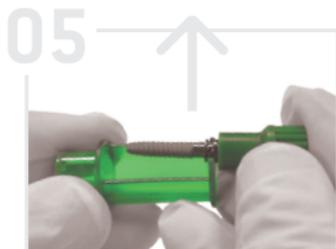

5. Remove the implant by vertically lifting upwards.



6. The implant can also be taken into the patient's mouth with the handpiece by directly inserting it in the metallic implant-holder.



7. Do the same motion as in fig. 5, remove by vertically lifting upwards.