


IMPLANTS with multifunction abutment PACKAGING



1. Open the implant threaded cap counter-clockwise.



2. Remove the aluminum cap from the container by a flap.

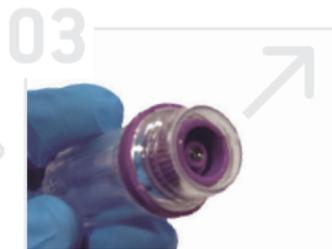

3. Once the cap has been removed, extract the implant's container. The plastic cap contains the cover screw.



4. Remove the plastic cap from the inner container.



5. Then, remove the implant with the manual key for multifunction abutment.



6. The implant can also be removed with the contra angle key for multifunction abutment.

Instructions for use
IMPLANTS with multifunction
abutment PACKAGING


MICRODENT
IMPLANT SYSTEM